**OPERATING INSTRUCTIONS AND SPECIFICATIONS FOR  
THE**

**70 WATT BIPOLAR ELECTROSURGICAL GENERATORS**

**28-2000 - 70W Coagulator**

**28-2600 - 70W Coagulator w/Irrigation**

## TABLE OF CONTENTS

English	1
Français	13
Deutsch	25
Italiano	37
Español	49
Português	61

## **English**

### **APPLICATION**

This manual applies to the AURA<sup>®</sup> 70 watt bipolar electrosurgical generators (Model Numbers 28-2000 and 28-2600) produced by Kirwan Surgical Products LLC

### **INDICATIONS**

The AURA<sup>®</sup> 70 watt bipolar electrosurgical coagulator is a general purpose solid state bipolar coagulator used to supply the RF signal to electrosurgical handpieces used on soft body tissues where a wide range of tissue types, patient conditions, and load impedances are encountered.

### **WARNINGS AND CAUTIONS**

This device is restricted to use by a qualified, licensed practitioner. Federal (USA) law restricts this device to sale by or on the order of a physician.

Read this entire manual prior to use of this product.

Do not remove cover. Refer servicing to qualified service personnel.

Do not use in the presence of flammable gasses.

Use of this device in the presence of patients with pacemakers or active implants is not recommended. Approved qualified advice should be obtained prior to any such use.

If the AURA<sup>®</sup> 70 is used simultaneously with any physiological monitoring equipment, monitoring electrodes should be placed as far from surgical electrodes as possible. Needle monitoring electrodes are not recommended. Monitoring systems incorporating high frequency current limiting devices are recommended.

Connect the generator power cord to a properly grounded receptacle. Do not use adapters or extension cords.

Connect only bipolar accessories to the bipolar receptacles. Improper connection of accessories may result in inadvertent accessory activation or other potentially hazardous conditions.

Use only accessories whose voltage rating is greater than the maximum output voltage of the AURA<sup>®</sup> 70 (210 V<sub>peak</sub>). Accessories, including cables, should be routinely inspected for any damage prior to use. Refer to the instructions for use of each individual accessory as needed.

Temporarily unused active accessories shall be stored in a location that is isolated from the patient. Accessory cables shall be positioned in such as way as to avoid contact with the patient or other leads.

The lowest possible output setting necessary to achieve the desired effect should be used at all times.

Special precautions regarding electromagnetic compatibility (EMC) should be exercised prior to installing and putting the AURA<sup>®</sup> 70 into service. Refer to technical service manual IFU-70WSERV-5 for further information.

The AURA<sup>®</sup> 70 complies with standard EMC requirements when used with bipolar handpieces up to 10.5 inches (26.7cm) in length and bipolar cables up to 12 feet (3.7m) in length produced by Kirwan Surgical Products LLC (refer to product catalog L-GENCAT-6 or [www.ksp.com](http://www.ksp.com) for additional accessory information). The use of longer handpieces and/or cables, or those produced by other manufacturers, could adversely affect the EMC performance of the unit.

The AURA<sup>®</sup> 70 produces high frequency energy that may adversely affect the operation of other electronic equipment in its vicinity.

Portable or mobile communications equipment used in the vicinity of the AURA<sup>®</sup> 70 could potentially cause slight disturbances in the function of the device.

Any use of this device for tasks other than for which it is indicated, may result in damaged equipment or possible injury to the user or patient.

## ESSENTIAL PERFORMANCE

The AURA<sup>®</sup> 70 shall not change output power settings during use. Unintended HF energization is not allowed.

## ON-SCREEN ERROR MESSAGES


<b>Color</b>	<b>Message</b>	<b>System Operation Stopped</b>	<b>Cause</b>	<b>Action</b>
Yellow	“Hot! Decrease Power”	No	Internal temperature of the unit exceeds 40°C	Decrease power setting.
Yellow	“Pump Stopped. Release pedal for reset”	No	Maximum pump motor current limit is reached	Release pedal.
Yellow	“Pump Error. Refer for service”	No	Problem with irrigation pump	Contact service.
Red	“Error. Refer for service. Code XX.”	Yes	Internal malfunction	Contact service. Reference "XX" error code.

## CONTROLS AND INDICATORS

**Figure 1**

- \* Pump Tube Slot - Accepts standard Kirwan irrigation cords
- \* Pump Module – Accepts silicone pump tube on standard Kirwan irrigation cords
- \* HF Out LED – Lit when bipolar output is activated
- \* Bipolar Output Receptacles – Accept standard 4mm banana pins
- \* Flow Rate Display – Displays flow rate setting of pump module in cubic centimeters per minute (cc/min)
- \* Flow Rate Adjust – Adjusts flow rate of pump module up or down
- \* Irrigation ON/OFF – Enables irrigation pump module ON in tandem with bipolar output
- \* Active/Inactive Display – Indicates status of bipolar output
- \* Power Level Display, Watts – Displays output power level setting in watts
- \* Power Level Adjust – Adjusts output power level up or down
- \* Time Display – Displays total elapsed time bipolar output is activated during procedure
- \* Reset – Resets time display to 00:00
- \* Options – Accesses controls for screen brightness, speaker volume, and tube priming
- \* Power Level Display, Neuro – Displays output power level setting in Neuro units

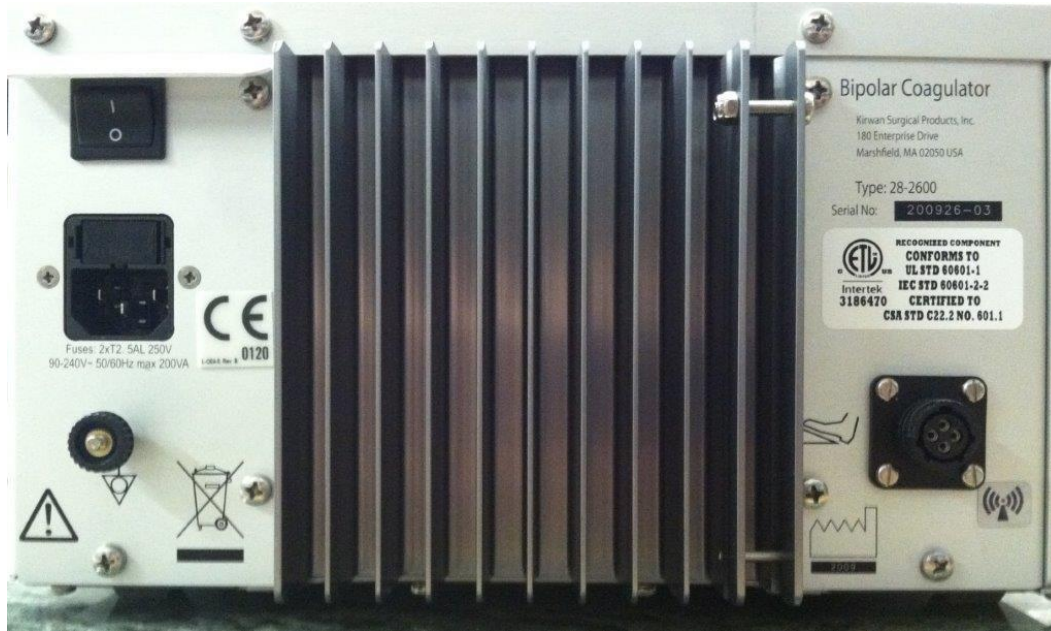**Figure 2**

- \* Power On/Off Switch
- \* Fuse Holder
- \* Mains Power Cord Connector
- \* Equipotentiality Connection
- \* Slot for IV Pole – Located at top right corner of heat sink
- \* Footswitch Connector

## SETUP


Connect the power cord to the power cord connector on the rear panel of the unit. Ensure the main power switch is in the off (0) position prior to connecting the cord.

Connect the footswitch to the footswitch connector on the rear panel of the unit.

Connect the banana pins of the bipolar cord into the bipolar output connectors.

If irrigation is to be used, refer to the figure below and load the silicone pump tube as follows:

- Form the silicone pump tube into a semi-circle and loop it around the roller wheel of the pump module, proximal end (closest to the spike) on top.
- Pull the plastic connectors on either end of the pump tube along the slots toward the rear of the unit, until the flanges on the connectors can be secured into place in the recesses located at the midpoint of the slots. Avoid over-stretching the tube.
- Insert the spike at the proximal end of the irrigation line into the appropriate container.

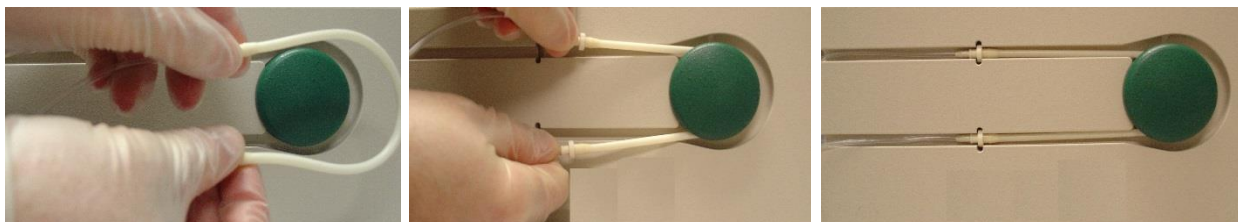**Figure 3 – Loading Silicone Pump Tube**

Connect the appropriate bipolar handpiece to the instrument end of the bipolar cord. Refer to the specific instructions for use provided with the accessory before using.

## **OPERATION**

Turn the unit on via the main power switch on the rear panel. After a brief diagnostic routine, the unit should come up in the state shown in Figure 1 (flow and power will be set to 00).

To adjust the volume level of the audible tone:

- Touch the **OPTIONS** button on the main screen
- Touch the **SOUND VOLUME** adjustment button and slide it left or right as needed, or touch the right or left arrows to increase or decrease the volume.
- Touch the **EXIT** button to return to the main screen

Adjust the output power level to achieve the desired effect by touching the power level adjustment arrows on the display screen. The power level will be displayed simultaneously in both watts and Neuro units.

Depress the footswitch pedal to activate the bipolar output to the accessory handpiece. An audible tone will sound indicating that the bipolar output is activated. Units will also provide visual indication via a blue “Active” indicator on the screen.

To reset the elapsed time display to 00:00, press the **RESET** button.

## **IRRIGATION OPERATION (Model 28-2600 only)**

To prime the irrigation line with irrigating solution:

- Touch the **OPTIONS** button on the main screen
- Touch the **PRIME** button. The pump motor will run continuously while this button is touched. Observe distal end of handpiece for presence of irrigant.
- Touch the **EXIT** button to return to the main screen

Adjust the flow rate as necessary by touching the flow rate adjustment arrows on the display screen. Note - Flow rates indicated on-screen are approximations. Variations in tubing durometer, solution container types, etc., may cause variations in actual flow output rates.

Touch the **Irrigation ON/OFF** button on the screen in order to allow flow of irrigating solution to the accessory handpiece via footswitch activation.

To return to activation of the bipolar output only, touch the **Irrigation ON/OFF** button again.

## SPECIFICATIONS

Input: 90-240VAC, 50/60Hz, max 200VA

Class: I

Applied Part: Type BF, Defibrillation-proof

Output: 512KHz, max 70W at 100-300 ohms load

Rated load: 200 ohms

Maximum output voltage: 210 V<sub>peak</sub>

Fuses: 2 x T2.5A 250V

Weight: 15 lbs. (6.8 kg)

Footprint: 11 ½ in W x 7 in H x 12 ½ in D (height does not include IV pole)  
(29.2 cm x 17.8 cm x 31.8 cm)

## EXPLANATION OF EXTERNAL MARKINGS

Consult the Instructions for Use (label)

Warning! Dangerous voltage (front panel)

Defibrillator-proof type BF applied part (front panel)

High Frequency current output (front panel)

Caution! Consult Accompanying Documents (rear panel)

Footswitch connection (rear panel)

Non-ionizing radiation (rear panel)


Date of Manufacture (rear panel)

Equipotentiality (rear panel)

Any accessories and all replaced parts must be disposed of and recycled in accordance with local environmental regulations regarding the disposal of used equipment and waste (rear panel)

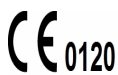CE Mark

## MAINTENANCE

### CLEANING INSTRUCTIONS

Use a mild detergent and damp cloth to clean the generator cover, power cord, and footswitch. In the presence of significant amounts of splashed liquids, protecting the footswitch with a disposable plastic bag may be desirable in order to simplify cleaning and ensure optimum performance of the switch over the life of the product.

Use only a soft damp cloth to clean the touch screen display. No chemicals should be used. Clean the display only when the unit is turned off to avoid any inadvertent activation.

The generator and all components must be completely dry before use.

Do not sterilize the generator.

### PREVENTIVE MAINTENANCE

At least once per year, the following inspections should be performed:

Visually check the condition of the AC power cord and footswitch cord (fraying of cables, loosening of connector ends, damaged insulation or contact pins, etc.). See the service parts list below if replacements are required.

Test the actual output power of the unit compared to the front panel setting, and the RF leakage current to ground, via an ESU analyzer or other appropriate method. The measured output power should not vary from the set power by more than 20%. The RF leakage current should not exceed 30mA. If the unit fails to comply with either of these requirements, it should be returned to Kirwan Surgical Products LLC as per the last page of this document for further evaluation.

### SERVICE/ACCESSORY PARTS LIST



<u>Description</u>	<u>Part Number</u>
Fuse, T2.5AL/250V	28-2000-70
Detachable Power Cord, Hospital Grade, 12 ft (3.7m)	26-2535
Square Pedal Footswitch w/ 15 ft (4.6m) Cord & 4 Pin Connector	28-2000-61
IV Pole	28-2600-40
Testing & Service Manual	IFU-70WSERV-5

All other components of this product are NOT field-replaceable. Contact Kirwan Surgical Products LLC as per the last page of this document if servicing of other components is required.

## TROUBLESHOOTING GUIDE

Symptom	Possible Cause(s)	Correction
Unit does not power up	Power cord not plugged in correctly	Check power cord connection.
	Blown fuse(s)	Check/replace fuse(s)
	Bad power cord	Check/replace power cord
	Fault in internal power supply	Refer to qualified personnel for servicing
No RF output	Bipolar cord not plugged in correctly	Check bipolar cord-ESU connection
	Instrument not plugged into bipolar cord correctly	Check instrument-bipolar cord connection
	Footswitch cord not plugged in correctly	Check footswitch cord connection.
	Faulty footswitch	Check/replace footswitch
	Bad bipolar cord	Check/replace bipolar cord
	Eschar build-up on instrument tip	Check/clean instrument tip
	Fault in internal circuitry	Refer to qualified personnel for servicing
No irrigation flow	Irrigation tube not installed correctly	Check irrigation tubing installation
	Leak or blockage in irrigation tubing	Check/replace bipolar irrigation cord
	Debris in pump roller assembly	Refer to qualified personnel for servicing
	Fault in pump motor assembly	Refer to qualified personnel for servicing
LCD Display Screen goes blank	Excessive Electrostatic Discharge (ESD) from operator to screen	Cycle Power switch off and on again. Refer to qualified personnel for servicing if necessary
RF power drops unexpectedly	Transient Voltage on Mains Line	Release footswitch pedal and depress again
	Eschar build-up on instrument tip	Check/clean instrument tip
Touch screen not functioning	Software error	Cycle Power switch off and on again. Refer to qualified personnel for servicing if necessary
	Fault in screen and/or other internal circuitry	Refer to qualified personnel for servicing
LCD Display too dim	Incorrect brightness/contrast adjustment	Adjust brightness/contrast control
	Fault in internal circuitry	Refer to qualified personnel for servicing
Audible tone too low	Incorrect volume adjustment	Adjust volume control
	Fault in internal circuitry	Refer to qualified personnel for servicing

# OUTPUT WAVEFORMS

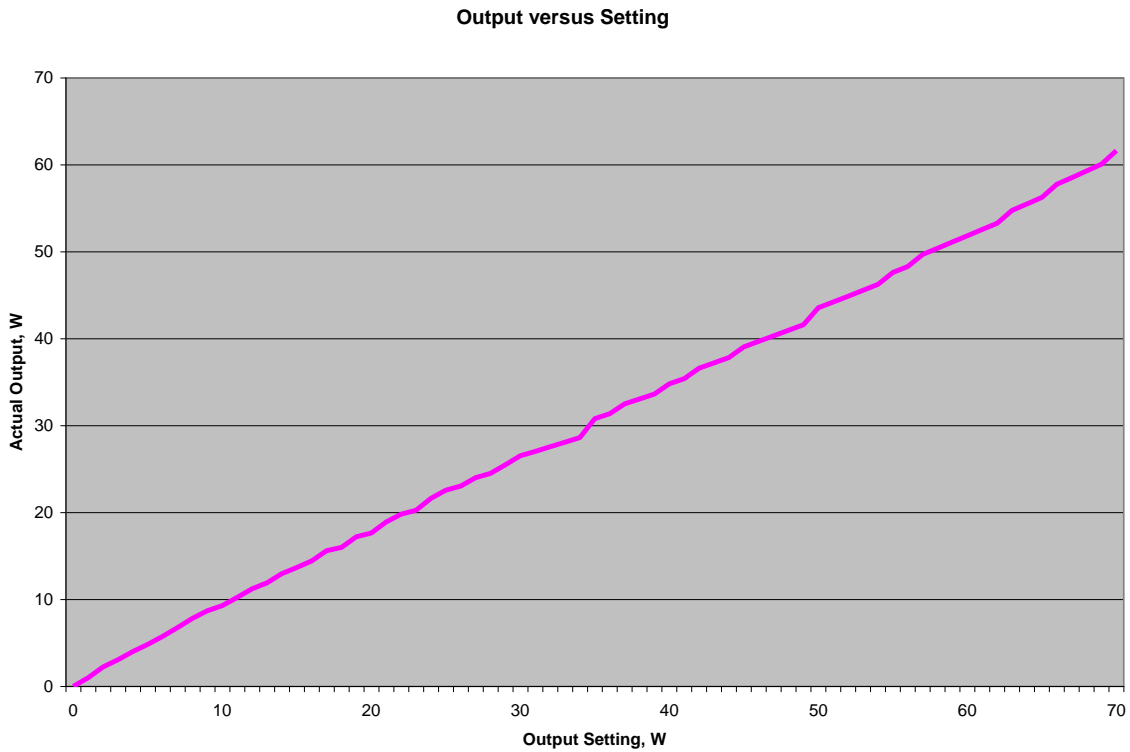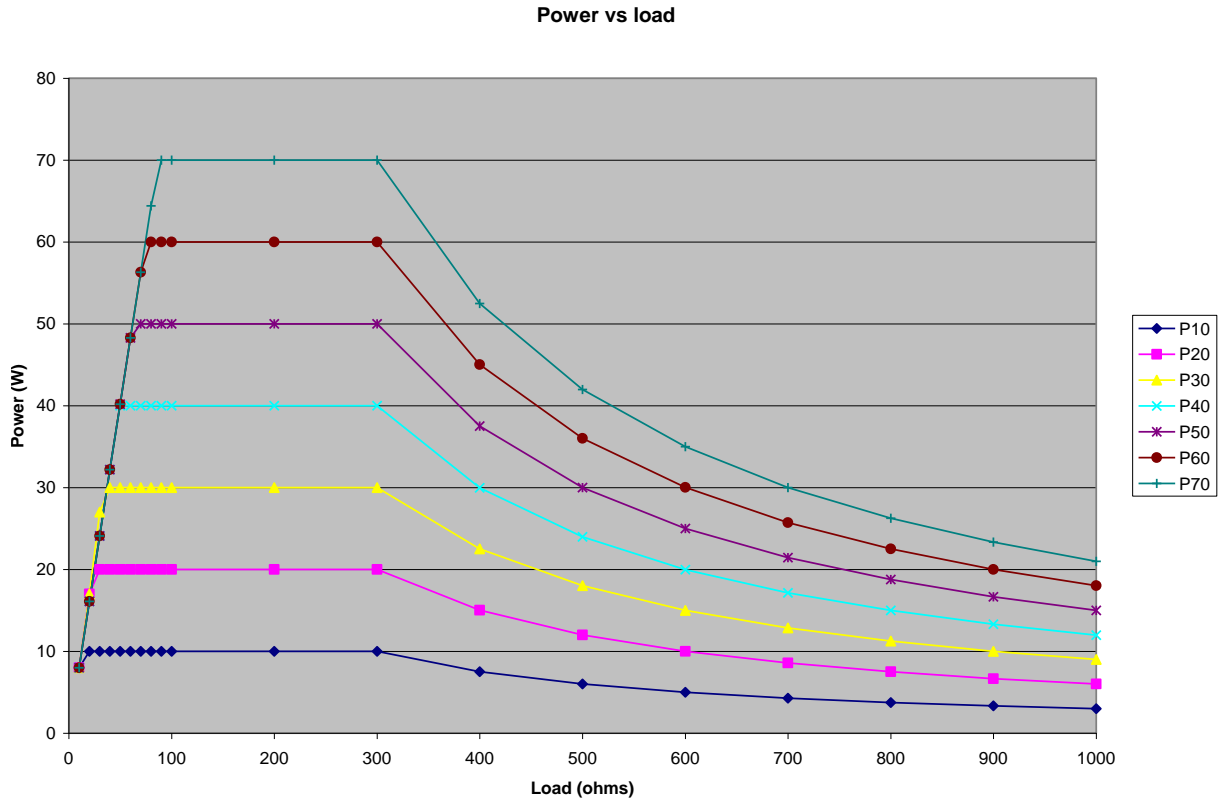**CONVERSION CHART – WATTS vs. “Neuro” UNITS**

Watts	"Neuro" Units
1	9
2	17
3	20
4	23
5	25
6	27
7	30
8	32
9	35
10	37
11	39
12	41
13	43
14	45
15	47
16	49
17	51
18	55
19	59
20	63
21	67
22	71
23	76
24	81
25	89
26	92
27	95
28	98
29	100
30	104
31	107
32	110
33	113
34	116
35	119

Watts	"Neuro" Units
36	122
37	124
38	126
39	129
40	131
41	133
42	134
43	136
44	138
45	140
46	142
47	144
48	146
49	148
50	150
51	152
52	154
53	156
54	158
55	159
56	161
57	163
58	165
59	167
60	169
61	171
62	173
63	175
64	177
65	179
66	181
67	183
68	184
69	186
70	188

## **WARRANTY INFORMATION**

### **WARRANTY**

This product is guaranteed to be free from defects in material and workmanship in normal use in surgery. If the product breaks or fails to function in normal use due to defects in materials or workmanship, within one year of purchase, Kirwan Surgical Products LLC will repair or replace such equipment at no cost to the customer after an evaluation has been made.

The above warranty excludes modified or altered products, and applies only to the original buyer and is in lieu of all other warranties either expressed or implied.

### **LIMITATIONS OF CLAIMS**

No claim of any kind, whether as to materials delivered or for non-delivery of materials shall be greater in amount than the purchase price of the materials in respect of which damages are claimed, and in addition, SELLER shall have no liability whatsoever for consequential damages, including, but not limited to, damages for injuries to persons or property. Claims for non-delivery of shortages must be submitted in writing within 30 days of date of the invoice. Claims for damaged material must be submitted within 30 days of the date of shipment. Deductions for late claims shall be disallowed.

### **OPEN SOFTWARE**

This product includes software code developed by third parties, including software code subject to the GNU General Public License ("GPL") or GNU Lesser General Public License ("LGPL"). As applicable, the terms of the GPL and LGPL, and information on obtaining access to the GPL Code and LGPL Code used in this product, are available to you by e-mailing a request to [sales@ksp.com](mailto:sales@ksp.com), or by calling the telephone numbers listed at the end of this document.

The GPL Code and LGPL Code used in this product is distributed WITHOUT ANY WARRANTY and is subject to the copyrights of one or more authors. For details, see the GPL Code and LGPL Code for this product and the terms of the GPL and LGPL.

## RETURN OF UNUSED EQUIPMENT POLICY

1. There is no restocking charge for unopened products returned within 30 days as a result of order error, if a replacement is ordered. Otherwise, there will be a 15% restocking charge based on the original purchase price.
2. To return unused equipment, in unopened packages, the BUYER must request a RETURN AUTHORIZATION NUMBER by calling customer service at the telephone numbers provided below. All returns must be accompanied by paperwork listing products to be returned, a copy of the purchase order and invoice, and return authorization number.
3. All returns must be freight prepaid and sent to:  
Kirwan Surgical Products LLC  
180 Enterprise Drive  
Marshfield, Massachusetts 02050  
USA  
  
Phone: (781) 834-9500  
1-888-547-9267 (Outside of MA)  
FAX: (781) 834-0022  
Web: [www.ksp.com](http://www.ksp.com)

**EC REP** For Regulatory Issues Only  
Medical Product Service GmbH  
Borngasse 20  
35619 Braunfels, Germany

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