**KIRWAN SURGICAL PRODUCTS LLC  
INSTRUCTIONS FOR USE –  
REUSABLE BIPOLAR ADAPTER**

This product is reusable and is supplied non-sterile.

**INDICATIONS:** The enclosed adapter is intended to be used to convert a small pin to a standard banana pin.

**CONTRAINDICATIONS:** Any use of this instrument for tasks other than for which it is indicated will usually result in a damaged or broken instrument.

**SETUP AND USE**

**WARNING:** CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE AND MONOPOLAR ACCESSORIES TO THE MONOPOLAR RECEPTACLE ONLY. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

Connect the pins from the cord or device to the receptacle end of the adaptor. Then connect the adapter to the electrosurgical generator. Do so only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

While attaching the adapter to a device or cord, ensure that the contact pins are fully seated in the receptacles.

**CLEANING**

1. Remove adapter from generator and disconnect any attached cord.
2. Prepare a neutral pH detergent or enzymatic detergent according to the detergent manufacturer's directions.
3. Use a soft, clean cloth lightly moistened with the cleaning solution to manually clean all surfaces of the adapter.
4. Rinse thoroughly and dry with a soft, clean cloth.

Power setting guidelines may vary due to difference in surgical techniques, patients, electrodes and surgical set-up. Start at the lowest power setting and increase as necessary to achieve the desired clinical effect.

**CAUTION:** BECAUSE OF THE VARIABILITY OF OUTPUT VOLTAGES AND MODES FROM GENERATOR TO GENERATOR, DO NOT USE THE ADAPTER WITH GENERATOR SETTINGS HAVING A VOLTAGE EXCEEDING 1100 V<sub>p-p</sub>. REFER TO THE APPROPRIATE ELECTROSURGICAL GENERATOR MANUAL FOR INDICATIONS AND INSTRUCTIONS ON VOLTAGE OUTPUT CHARACTERISTICS TO ENSURE THAT ALL SAFETY PRECAUTIONS ARE FOLLOWED.

CONSULT INSTRUCTIONS FOR USE

CATALOG NUMBER

BATCH CODE


DATE OF MANUFACTURE

NON STERILE

NOT MADE WITH NATURAL RUBBER LATEX  
PRODUCT DOES NOT CONTAIN LATEX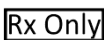

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician

Kirwan Surgical Products LLC  
180 Enterprise Drive  
Marshfield, MA 02050, USA  
Tel: (781) 834-9500 www.ksp.com  
Made in USA

Medical Product Service  
Borgasse 20  
35619 Braunfels, Germany

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