**KIRWAN SURGICAL PRODUCTS LLC**  
**INSTRUCTIONS FOR USE**  
**DISPOSABLE IRRIGATION BIPOLAR CORD**

This product is disposable and is supplied **STERILE**. Prior to the expiration date, contents are sterile if the package has not been opened or damaged

**10-4601 - INTENDED USE:** The Disposable Irrigation Bipolar Cord is designed to be compatible with the Kirwan AURA™ Irrigating Generator as well as the Malis™ CMC® Bipolar System and similar bipolar irrigating systems.

**10-4602 and 10-4602J – INTENDED USE:** The Disposable Irrigation Bipolar Cord is designed to be compatible with Malis™ 1000 Irrigating Bipolar systems.

**CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.**

**INTENDED USE:** The Disposable Irrigation Bipolar Cord is an electrosurgical accessory designed to be used with standard bipolar irrigating forceps.

**NOTE:** This product is designed for use with collapsible irrigation solution containers only, and for use with bipolar electrosurgical units only.

**NOTE:** DO NOT USE THIS PRODUCT IF STERILE PACKAGE IS DAMAGED OR SEAL IS BROKEN.

**SETUP:**

1. Close the roller clamp on the tubing.
2. Remove the cap from the spike and insert into the collapsible irrigation solution container.
3. Place the irrigation solution container onto an I.V. pole.
4. Insert the section of silicone tubing into the irrigation pump module if one is used refer to pump manufacturer's instruction.
5. Remove the luer cap and connect the male luer connector of the Irrigation Cord to the bipolar forceps.
6. Connect the Irrigation Cord plug to the bipolar forceps.
7. Insert the banana pins into the bipolar generator.

**WARNING:** Connect Bipolar accessories to the Bipolar receptacle only, and Monopolar accessories into the Monopolar receptacle. Improper connection of accessories may result in inadvertent accessory activation or other potentially hazardous conditions.

**WARNING:** Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the application of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.

**WARNING:** Do not reuse or re-sterilize. Patient cross contamination and/or product malfunction may occur.

**CAUTION:** *Because of the variability of output voltages and modes form generator to generator, DO NOT USE the Disposable Irrigation Bipolar Cord with generators having output voltages that exceed 1100Vp-p. Refer to the appropriate electrosurgical generator manual for indications and instructions on output characteristics to ensure that all safety precautions are followed.*


**CAUTION:** *The non-booted connection of this cord should be considered “NOT PROTECTED” against ingress of liquid with harmful effects (CODE IPX0).*

**PROPER DISPOSAL** of the irrigation cord possibly contaminated with blood, tissue, or other potentially infectious material present a biological risk and must be discarded in a closable, leak-proof, puncture-resistant receptacle, that is adequately labeled (e.g., color coding or symbology) for easy identification as biohazard waste.

AURA™ is a registered trademark of Kirwan Surgical Products LLC.

Malis™ is a trademark of Synergetics IP, Inc.

CMC is a registered trademark of Johnson & Johnson Professional, Inc.

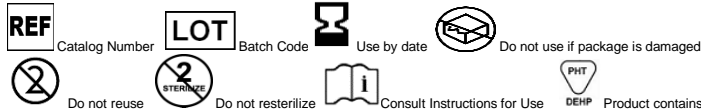**Rx Only** **CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.**

CE 0120

Kirwan Surgical Products LLC  
 180 Enterprise Drive  
 Marshfield, MA 02050, USA  
 Tel: (781) 834-9500

**EC REP** For Regulatory Issues Only  
 Medical Product Service GmbH  
 Borngasse 20  
 35619 Braunfels, GERMANY

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AURA™ é uma marca comercial registada da Kirwan Surgical Products LLC.
Malis™ é uma marca comercial da Synergetics IP, Inc.
CMC é uma marca comercial registada da Johnson & Johnson Professional, Inc.