ENGLISH

KIRWAN SURGICAL PRODUCTS LLC INSTRUCTIONS FOR USE – REUSABLE MONOPOLAR ADAPTER

This product is reusable and is supplied non-sterile.

INDICATIONS: The enclosed adapter is intended to be used to convert a resectoscope pin to a standard banana pin or Bovie®-type pin, or a standard banana pin to a Bovie® type pin.

CONTRAINDICATIONS: Any use of this instrument for tasks other than for which it is indicated will usually result in a damaged or broken instrument.

SETUP AND USE

WARNING: CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE AND MONOPOLAR ACCESSORIES TO THE MONOPOLAR RECEPTACLE ONLY. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

Connect the pin from the cord or device to the receptacle end of the adapter. Then connect the coagulator and cable to the electrosurgical generator. Do so only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

While attaching the adapter to a device or cable, ensure that the contact pins are fully seated in the receptacles.

CLEANING

1. Remove adapter from generator and disconnect any attached cable.
2. Prepare a neutral pH detergent or enzymatic detergent according to the detergent manufacturer's directions.
3. Use a soft, clean cloth lightly moistened with the cleaning solution to manually clean all surfaces of the adapter.
4. Rinse thoroughly and dry with a soft, clean cloth.

Power setting guidelines may vary due to differences in surgical techniques, patients, electrodes and surgical setup. Start at the lowest power setting and increase as necessary to achieve the desired clinical effect.

CAUTION: BECAUSE OF THE VARIABILITY OF OUTPUT VOLTAGES AND MODES FROM GENERATOR TO GENERATOR, DO NOT USE THE MONOPOLAR ADAPTER WITH GENERATOR SETTINGS HAVING A MONOPOLAR VOLTAGE EXCEEDING 7000 Vp-p. REFER TO THE APPROPRIATE ELECTROSURGICAL GENERATOR MANUAL FOR INDICATIONS AND INSTRUCTIONS ON VOLTAGE OUTPUT CHARACTERISTICS TO ENSURE THAT ALL SAFETY PRECAUTIONS ARE FOLLOWED.

Manufactured by:
Kirwan Surgical Products LLC
180 Enterprise Drive
Marshfield, MA 02050

Authorized EU Representative:
Medical Product Service
Borgasse 20
35619 Braunfels, Germany

Catalog Number

CE1639

Batch Code

Date of Manufacture

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician

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Fabricant:
Kirwan Surgical Products LLC
180 Enterprise Drive
Marshfield, MA 02050

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