**KIRWAN SURGICAL PRODUCTS LLC
INSTRUCTIONS FOR USE & CARE
MONOPOLAR DISPOSABLE TUNGSTEN
MICRODISSECTION NEEDLES – STERILE**

This product is disposable, intended for single use only and is supplied sterile. Contents are sterile if the package has not been opened or damaged.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

INTENDED USE: The monopolar disposable tungsten microdissection needle is an electrosurgical device designed to be used in soft tissue surgical procedures.

SETUP and USE:

WARNING: CONNECT MONOPOLAR ACCESSORIES TO THE MONOPOLAR RECEPTACLE ONLY. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

WARNING: DO NOT REUSE OR RE-STERILIZE. PATIENT CROSS CONTAMINATION AND/OR PRODUCT MALFUNCTION MAY OCCUR.

CAUTION: Handle with care. Do not use forceps or pliers to place or remove the needle from a handpiece. Perforation or other damage can occur in the insulation, which can lead to unwanted tissue damage during use.

Take extreme care that the needle shaft is loaded properly in the hand piece and is secure. To prevent injury to the patient or operator, the insulation should only be visible at the end of the hand piece connection (not the bare needle shaft).

CAUTION: Needle tips are sharp. Use care when removing from pouch.

Load the needle into the hand piece only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

WARNING: Monopolar Disposable Tungsten Microdissection Needles require very low wattage outputs to perform most required cutting and coagulating. They are intended for Low Voltage Settings where the peak voltage does not exceed 4000 volts.

Begin procedures at the lowest possible power setting (approximately 3 – 5 Watts), slowly increasing power to achieve desired cut. Use only enough power to prevent drag and tissue build up.

SEE INSTRUCTIONS FOR USE

USE BY DATE

SINGLE PATIENT USE


BATCH CODE

CATALOG NUMBER

STERILE – STERILIZED BY IRRADIATION Sterility guaranteed unless package opened or damaged. Do not resterilize.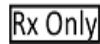

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

NO LATEX

NOT MADE WITH NATURAL RUBBER LATEX

Kirwan Surgical Products LLC
180 Enterprise Drive
Marshfield, MA 02050, USA
Tel: 1-888-547-9267
Made in USA

Medical Product Service
Borngasse 20
35619 Braunfels Germany

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