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ENGLISH

KIRWAN SURGICAL PRODUCTS LLC INSTRUCTIONS FOR USE REUSABLE FORCEPS STERILIZATION TRAY

This product is reusable and is supplied non-sterile.

INDICATIONS: The enclosed sterilization tray is intended to be used to hold reusable bayoneted forceps during autoclave sterilization

CONTRAINDICATIONS: Any use of this tray for tasks other than for which it is indicated, will usually result in a damaged or broken instrument.

SETUP and USE:

Ensure that the tray is clean prior to use. If not, clean per standard hospital protocol.

NOTE: High pH detergents may erode the anodized finish of the sterilization tray. In order to maintain the appearance of the tray's finish, a neutral pH detergent (pH level 6.5 – 8.0) is recommended.

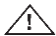
Examine the contents of the tray itself to see that all parts are present and in working condition (locks, retainers, pads). If the retainers or pads have worked free from the tray, press back into position. If the locks have become loose, retighten. If any parts are missing or broken do not use to sterilize forceps as this may cause damage to your forceps.

Disconnect the forceps to be sterilized from any connecting cables. Place forceps to be sterilized in the bottom tray. They should be adequately contained within the retainers and the tips should rest on top of the retaining pad. Do not place forceps on top of each other or allow them to contact other forceps during the sterilization process.

If the retainers will not correctly hold your device, do not use the tray for sterilization as this may cause damage to your device.

Place the tray cover over the tray bottom. Engage the locks at both ends of the tray. Sterilize instruments per their specific requirements or standard procedure.

Please refer to the Kirwan Surgical Products LLC catalog for our complete line of electro-surgical devices and for replacement parts availability.

	SEE INSTRUCTIONS FOR USE
REF	CATALOG NUMBER
	BATCH CODE
	DATE OF MANUFACTURE

	Kirwan Surgical Products LLC 180 Enterprise Drive Marshfield, MA 02050, USA Tel: 1(781) 834-9500 MADE in USA
EC REP	Medical Product Service Borngasse 20 35619 Braunfels, GERMANY

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KIRWAN SURGICAL PRODUCTS LLC INSTRUÇÕES DE UTILIZAÇÃO – CAIXA DE ESTERILIZAÇÃO DE PINÇAS REUTILIZÁVEIS