DEUTSCH

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 Medical Product Service  
Borngasse 20  
35619 Braunfels, GERMANY

ENGLISH

KIRWAN SURGICAL PRODUCTS LLC  
INSTRUCTIONS FOR USE & CARE -  
DISPOSABLE BIPOLAR FORCEPS

This product is disposable and is supplied sterile. Prior to the expiration date, contents are sterile if package has not been opened or damaged.

**WARNING:** Do not reuse or re-sterilize. Patient cross contamination and/or product malfunction may occur.

**INTENDED USE:** Disposable forceps are electrosurgical devices designed to be used in soft tissue procedures.

**SETUP AND USE**

Attach the sterile forceps to the sterile cable ensuring that the forceps pins are fully seated in the cable receptacles. This condition ensures that the connection is splash proof. The cables to the surgical electrodes should be positioned in such a way that contact with the PATIENT or other leads is avoided. Temporarily unused ACTIVE ELECTRODES should be stored isolated from the patient.

Connect the sterile cable to the generator only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

**CAUTION:** BECAUSE OF THE VARIABILITY OF OUTPUT VOLTAGES AND MODES FROM GENERATOR TO GENERATOR, DO NOT USE THE FORCEPS WITH GENERATOR SETTINGS HAVING BIPOLAR OUTPUT VOLTAGES THAT EXCEED 1100 Vp-p. REFER TO THE APPROPRIATE ELECTROSURGICAL GENERATOR MANUAL FOR INDICATIONS AND INSTRUCTIONS ON BIPOLAR OUTPUT CHARACTERISTICS TO ENSURE THAT ALL SAFETY PRECAUTIONS ARE FOLLOWED.

**WARNING:** CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE ONLY, AND MONOPOLAR ACCESSORIES INTO THE MONOPOLAR RECEPTACLE. IMPROPER CONNECTOR OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

Power setting guidelines may vary due to differences in surgical techniques, patients, electrodes and surgical set-up. Start at the lowest power setting and increase as necessary to achieve the desired clinical effect.

At the lowest power setting, test the forceps by pressing the generator's activating switch. If the generator fails to activate, check the forceps connection with the cable.

 SEE INSTRUCTIONS FOR USE

 USE BY:  
YEAR - MONTH

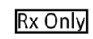
 SINGLE PATIENT USE

 BATCH CODE

 CATALOG NUMBER

 STERILE - STERILIZED BY IRRADIATION

Sterility guaranteed unless package opened or damaged. Do not resterilize.

 CAUTION: Federal (USA) law restricts this device to Sale by or on the order of a physician.

  Do not resterilize.

 Kirwan Surgical Products LLC  
180 Enterprise Drive  
Marshfield, MA 02050, USA  
Tel: 1-888-547-9267  
MADE IN USA

 Medical Product Service  
Borngasse 20  
35619 Braunfels, GERMANY

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